

\*This flowchart is intended for University faculty, staff, students, and contractors. \*If you experience any symptoms, please contact the University or visit the nearest hospital. \*#7119 is a call center which provides access to expert advice from medical professionals. (JP only)



\*This flowchart is intended for University faculty, staff, students, and contractors.

\*If you experience any symptoms, please visit the nearest hospital.

\*#7119 is a call center which provides access to expert advice from medical professionals. (JP only)





\*This flowchart is intended for University faculty, staff, students, and contractors. \*If you experience any symptoms, please visit the nearest hospital.

\*#7119 is a call center which provides access to expert advice from medical professionals. (JP only)

## HEARTSTART HS1 QUICK GUIDE PHILIPS Overview PULL the green handle (turn on HS1) ① PULL the green handle **X** According to JRC G2010. if the victim is preschooler, 2 PLACE the pads use the Infant/Child pads.(Option) "Begin by removing clothing patients chest, cut clothing if needed." Analyze PLACE the pads on the patient's bare skin Shock is Shock is NOT needed needed Charge **③PRESS** the shock button "Do not touch the patient. Analyzing." 30 compressions & 2 breathings (2min)(If a shock is needed) (If a shock is NOT needed) "Shock not advised" "Shock advised, By user "Be sure EMS have been called" Stay clear of patient" By AED "It is safe to touch the patient" "Begin CPR" PRESS the flashing orange Shock button. SHOCK Start CPR in 2 min.(Cycles of 30 compressions and 2 breathings) \* If you cannot do artificial breathing, continue compressions. "Press the flashing orange" "Shock delivered" "Be sure EMS have been called" "It is safe to touch the patient" "Begin CPR." "For help with CPR, press blue." %Leave AED on until EMS arrive ! Do not remove the pads from the patient !